


#	Component Name
1	Mini cradle knob
2	MC- AVP Press Plane
3	AVP Connection

- A Release the Mini cradle Knob by rotating it counterclockwise
- B Align the MC- AVP Press Plane on top of the AVP Connection
- C Using 4 fingers, press the MC- AVP Press Plane on to the AVP connection until you hear the click. Make sure both sides of the MC- AVP Press Plane are fully inserted.

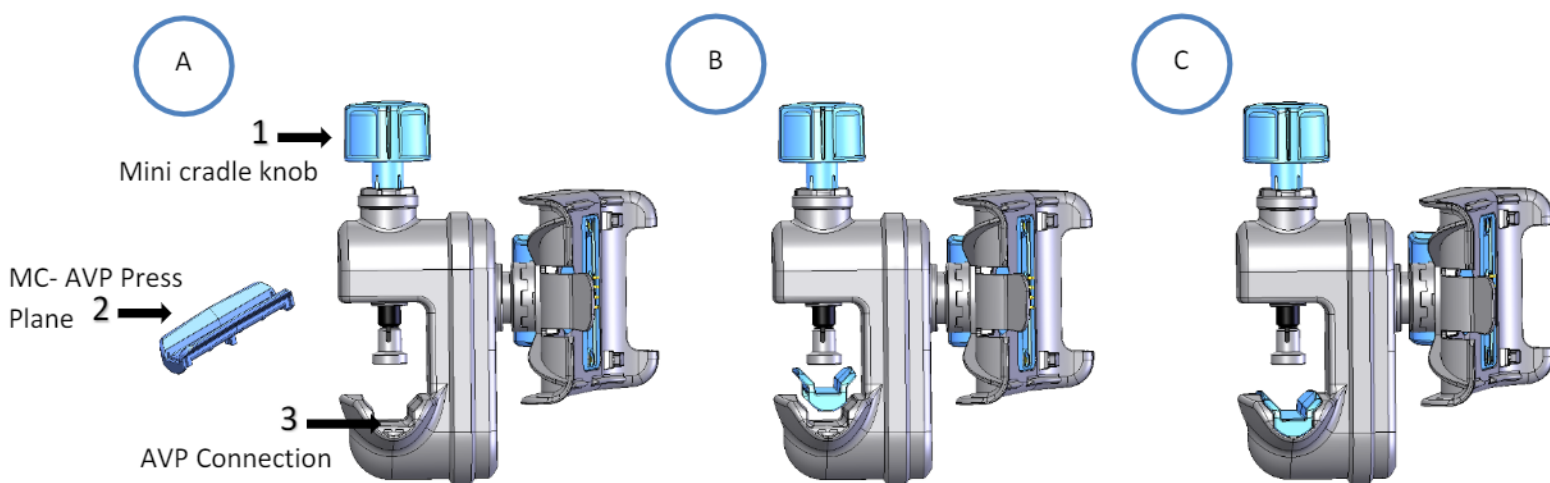

This is a representation of an electronic record that was signed electronically and this page is the manifestation of the electronic signatures.

Signatory Table

Action Name	User Name	Title	Signature Date
Send for Review (Written By)	Tal Ofer	Product Manager	12-Jan-2020 12:10 (GMT+3)
Send for Approval	Tal Ofer	Product Manager	20-Jan-2020 13:50 (GMT+3)
Approve	Lilach Tamir	Director of Product Management	22-Jan-2020 05:41 (GMT+3)
QA Approval QA Approve	Faina Manuvahov	QA Engineer	22-Jan-2020 18:41 (GMT+3)
Sign Training Completion	Faina Manuvahov	QA Engineer	26-Jan-2020 17:13 (GMT+3)